QUALITÄT | INNOVATION | SERVICE

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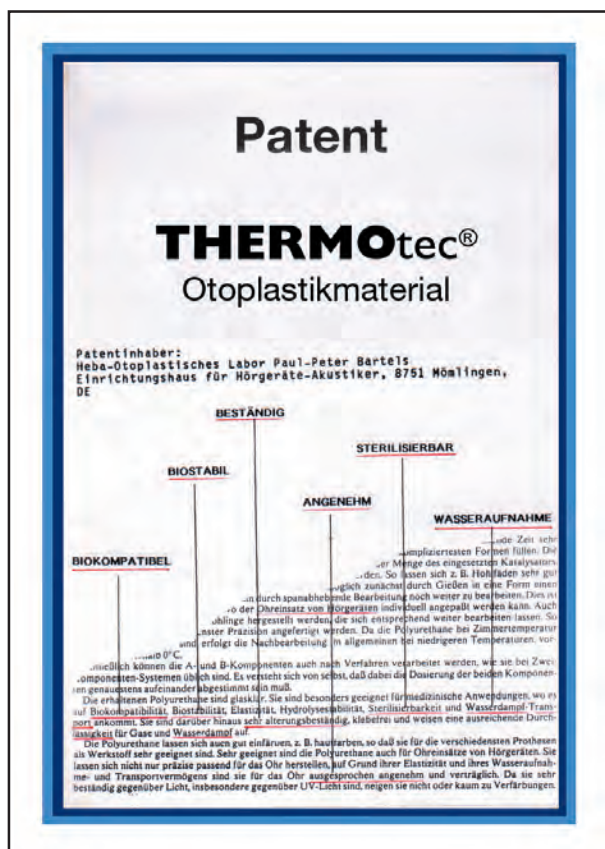
QUALITY | INNOVATION | SERVICE

In 1962 Paul Peter Bartels, a dentist, manufactured the first otoplastic for a friend, who was a hearing aid distributor. This was also the beginning of the company HEBA-OTOPLASTIK. Its aim has been to develop and produce a complete and integral coordinated system for hearing aid distributors.

HEBA – a competent and fair partner for you

The creative implementation of ideas, a professional handling of materials and state-of-the-art technology lead to recognized high-quality products.

Our daily work requires great effort due to more competition, higher customer expectations, and ever more complicated matters to solve, because everybody who suffers from a hearing impairment has very personal problems for which he needs a solution.

Successful together

We place great importance in quality and customer-oriented service. We develop innovative products for individual applications.

Patents and copyright designs, such as THERMOTec®, the otoplastic material, which is nowadays well-known all over the world, and the drying cushion HEBA®-THERM document this on an international scale.

Quality management

We assume, that quality assurance is a essential part of our daily work, and of course we got a certification under DIN EN ISO 13485 and the Medical Product Directive 93/42/EEC. So you are on the safe side too.

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article	page	article	page	article	page
A+B impression material, kneadable.....	48/50	Dosing spoon.....	48/50	HEBA-FORM A+B.....	48
A+B soft impression material.....	48/49	Dosing syringes.....	48	HEBA-FORM CIC.....	49
Acoustic tube, PVC by the meter.....	36	Drills.....	82	HEBA-FORM KIDS.....	49
Acoustic tube, PVC curved.....	36/37	Drying and cleaning set PUROCLEAN®.....	40/43/44	HEBA-FORM K/S.....	50
Acoustic tube, Silicone by the meter.....	36	Drying capsules PUROCLEAN®.....	44	HEBA-Silc.....	78
Acrylic-material.....	69/70	Drying cushion HEBA®-THERM.....	41-43	HEBA glue for PVC tubes.....	78
Addition equipment.....	19	Drying set PUROCLEAN®.....	44	HEBA thinner for glue.....	78
Adhesive primer.....	78	Drying tub PUROCLEAN®.....	44	HEINE products.....	52/53
Air blower.....	41	Dry-tube.....	34/36/37	Horn tube.....	34
Alabaster-Plaster.....	62	Duplicating material.....	58	Hot polymerisate.....	69/70
Aqua.....	49/73	Duplicating silicone.....	59	Hucki + Clip.....	38
Aquafit.....	95	Dust extraction system.....	89/90	Hucki — refill kit.....	38
				Hucki — retainer kit.....	38
Bakke-Horn.....	34	Ear light LUXAMED®.....	54	Hygienic article.....	40-46
Ball burs.....	82	Ear light MINI C.....	53		
Base parts for cuff forms.....	58/59	Ear-Classic soft.....	94	Impression extractor.....	80
Bayonett tweezer.....	52/80	Earmolds.....	10-17/24-26	Impression material.....	48-50
Biopact Hearing protection.....	30	EarPlanes.....	95	Impression pads.....	52
Black foils.....	64	Earplug Quiet.....	95	Impression pencil.....	52
Bottom plate for Micropol.....	62/72	Ears.....	52/100	Impression syringe.....	51
Box for membrane foils for Micropol.....	62/72	EIM-15 receiver for ELACIN®-earmold.....	32/96	In-ear-monitoring.....	32/96
Brass flask.....	62/74	ELACIN® hearing protection.....	29-31	Inject-gun for double cartridges.....	50/74
BTE-support earmold.....	14	ELACIN® ClearSound.....	31	ITE-shell material.....	64
Buffing wheel.....	85	Elbow.....	34		
		Embedding set for Micropol.....	62/72	Kids A+B impression material.....	49
Canula tips.....	50	Expanding ring.....	34		
Casting forms.....	58/59	Expanding ring seat.....	34	Lacquer.....	76/77
Casting material.....	59	Extractor.....	80	Lid for cutter box.....	86
Catamaran.....	49/73			Light curing units.....	65-68
Cleaning equipment.....	40-48	Face mask.....	97	Light tubes, blue light + UVA.....	68
Cleaning liquid.....	45	Face mask with micro filter.....	97	Linen buffing wheel.....	85
Cleaning powder HEBACLEAN®.....	40	Fill cylinder.....	51	Listening tube for ITE-hearing aids.....	35
Cleaning set HEBACLEAN®.....	40	Flex-liquid.....	70	LP/H — materials.....	64/65
Cleaning spray PUROCLEAN®.....	43	Flex-powder.....	70	LP-Cleaner.....	65
Cleaning tissues PUROCLEAN®.....	44	Floatable material.....	49/73	Luxamed® otoscope.....	53
Cleaning tub PUROCLEAN®.....	43	Foils.....	62/54/72	Luxamed® ear light.....	54
Clean-spray PUROCLEAN®.....	43	Forms for embedding.....	58/59		
ClearSound hearing protection.....	31	Funnels for otoscope.....	52-54	Machines.....	88/89
Click-System-Earmolds.....	16			Magnetic ear.....	100
Closing ratchet for Micropol.....	62/72	Glitter.....	70	Mandrel.....	55/84
Cold polymerisate.....	69	Gloves.....	97	Mandrel for sandpaper.....	84
Color concentrate.....	70	Glue.....	77/78	Membrane foils for Micropol.....	62/72
Comfort-earmolds.....	10/11	Gold-plated.....	18	Metal earmold ring.....	35
Cotton buffing wheel.....	85	Goggles.....	97	Metal earmold clip.....	35
Couplings.....	34	Grinding caps.....	85	Metal expanding coupling.....	34
Cuff forms.....	58/59	Grinding cap holder.....	85	Micropol 100.....	62/72
Cutter box.....	86	Grinding machines.....	88/89	Micro-tube otoplastik.....	16/17
Cutter machines.....	88/89	Grinding paper.....	84	Mini-Canulas.....	49
Cutters.....	82/83	Gun for double cartridges.....	50/74	MINI 3000 — otoscope.....	52
Cuvette.....	62/74			MINI C — ear light.....	53
Cyanacrylate glue.....	77	Hardener paste for HEBA-FORM K/S.....	50	MINI NT battery charger set.....	53
		Hearing protection earmolds.....	29-31	Mini oven.....	74
Decoration.....	100	Hearing protection for bikers.....	31	Mixing block.....	51
Demo ear.....	52/100	Hearing protection shotkiller®.....	30	Mixing canulas.....	50/74
Design earmolds.....	24-26	Hearing protection Standard.....	94/95	Mixing cup.....	71/81
Dipping wax.....	55	Heating plate.....	58	Mixing plate.....	51
Direct fitting material.....	74	HEBA®-THERM drying cushion.....	41-43	Mixing spatula.....	51/71/80/81
Display of hearing protection.....	96/100	HEBA®-THERM opti-TIME.....	42	Mixing spatula for plaster.....	80
Disposable funnels.....	53/54	HEBA-AQUA Catamaran.....	49/73	Model ear.....	100
Disposable gloves.....	97	HEBACLEAN® cleaning powder.....	40	Modeling knife.....	54/80
Dosing pump for silicone.....	59	HEBACLEAN® cleaning set.....	40	Modeling paste LP/H.....	65

article	page	article	page	article	page
Modeling set.....	68	Safety glass for Micropol.....	62/72	Tubing expander.....	83
Mold ring.....	35	Safety goggles.....	97	Turning motor for HEBA-LG 3.....	69
Monitoring.....	32/96	Sand paper.....	84	Tweezers.....	56/82
MultiFix tube.....	37	Scalpel (one way).....	54/81	Twist drills.....	84
NANOlac®.....	76	Scalpel blades.....	55/81	Ultra fit.....	96
Nylon rigid tube.....	34	Scalpel handles.....	57/81	Ultra Tech.....	96
Opti-TIME drying cushion.....	42	Scissor.....	52	Ultrasonic cleaner.....	49/50
O-ring for Micropol 100.....	62/72	Screw (spare part) for Micropol.....	62/72	Ultrasonic cleaner liquid.....	49
Oto-Ginkutan (Ginko Care).....	45	Separating fluid.....	62	UniFit®.....	98
Otoscope MINI 3000.....	52	Shotkiller® filter.....	30	UV-materials LP/H.....	66/67
Pads.....	52	Signing pencil.....	52	UV-technique.....	67-70
Perfecta 300/600.....	89	Silicone 40 Shore / 60 Shore.....	73	Valve inset for Micropol.....	64/74
Plaster.....	62	Silicone cutters.....	83	Valve screwdriver for Micropol.....	64/74
Polishing buffing wheel.....	85	Silicone dosing pump.....	59	Ventilette.....	94
Polishing motor.....	91	Silicone ears.....	100	Vibrating unit.....	92
Polishing paste.....	86	Silicone glue.....	77	Wax.....	58
Polishing powder.....	86	Silicone impression material.....	48-50	Wax heater, electrical.....	58
Polishing spindle.....	91	Silicone lacquer.....	76	Wax knife.....	57/82
Polishing trough.....	91	Silver-plated.....	18	Wax trap.....	39
Polymax.....	61/72	Sleep protection.....	28	Working environments.....	90-94
Polymerization unit.....	61/72	Soft canal tips (standard).....	37		
Polypot.....	61/71	Soft disc.....	55/84		
Power RIC-system.....	16	Spare acrylic top.....	53/54		
Pressure pot.....	61/71	Spare bulb.....	53		
Primer.....	78	Spare cylinder for impression syringe.....	51		
Pumice stone powder.....	86	Spare piston for impression syringe.....	51		
PUROCLEAN® cleaning set II.....	40	Spatula.....	51/71/80/81		
PUROCLEAN® cleaning tablets.....	41	Spindle for polishing motor.....	91		
PUROCLEAN® cleaning tissues.....	44	Splash-water protection.....	28/95		
PUROCLEAN® cleaning tub.....	43	Stetoclip.....	35		
PUROCLEAN® cleaning spray.....	43	Sticking wax.....	55		
PUROCLEAN® drying capsules.....	44	Superglue.....	77		
PUROCLEAN® drying set.....	44	Surface cleaner.....	65		
PUROCLEAN® drying tub.....	44	Surface refinement.....	19		
PVC acoustic tube.....	34	Technique machines.....	90/91		
Resimix-cup.....	71/81	Timer for HEBA®-THERM drying cushion.....	46		
Reusable funnels.....	52/54	Titanium.....	18/21		
RIC-systems.....	16/17	T-Mic holding ring by Advanced Bionics.....	14		
Rigid tube, by the meter.....	34	Tools.....	82-87		
Ring part for Micropol.....	62/72	Trimmer Doublespeed.....	94		
Rubber ear.....	52/100	Trimmer Singlespeed.....	93		
Rubber gasket for Micropol.....	62/72	Tube glue HEBA-SOFT for PVC tubes.....	80		
		Tube glue thinner for HEBA-SOFT.....	80		
		Tube spiral, winded.....	39		

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High frequency earmold

- improved high-frequency-transmission
- preservation of the natural ear canal resonance
- available in all shapes

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Foil earmold

- high wearing comfort
- elegant cosmetic
- available in all shapes
- improved heat conduction, no foreign body sensation
- THERMOTec® is particularly suited because of the thermo elasticity
- there is no feeling of pressure and the ear canal remains flexible

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Skid resistant shell

- foil-thin design
- distortion between the ear canal and the concha will be compensated
- tubing attached with gold-plated metal expanding coupling

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Standard shell

- compact design
- due to streamlined production methods no extras available

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Earmold

- for receiving the earphone of pocket devices
- with a metal or nylon expanding ring

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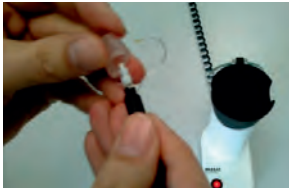
RIC-receiver and Micro-tube earmolds

The experience of recent years and the development of the market confirm the advantages to place the speakers in the ear canal. That's why modern RIC-receiver are increasingly being used.

To use the full potential of such systems, seating and positioning the receiver in the ear canal and the choice of the right earpiece is important for success. With innovation and practical ideas we would like to respond to your needs and those of your customers. In search of the optimal solution we will gladly advise you.

Our services:

- we produce RIC-receiver earmolds for all manufacturers
- available in all shapes (standard: foil plug with pull thread)
- with manufacturer-specific receiver couplings (if available from the manufacturer)
- hold through material — no socket (only THERMOtec®)
- if necessary processing with MultiFix-tube
- our thermo elastic material THERMOtec® has been excellent proven for RIC-receiver fittings over the years
- from open to Power, we offer solutions for every hearing loss

Remark:

Due to the minimal design of RIC-receiver and Micro-tube fittings, we recommend to produce the earmold with an inconspicuous short hook to prevent it from slipping out of the ear canal.

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HEBA-STARR, THERMOtec®, THERMOtec®-OPTIMUM

RIC-receiver earmold

- we produce RIC-receiver earmolds for all manufacturers
- available in all shapes (standard: foil plug with pull thread)
- with manufacturer-specific receiver couplings (if available from the manufacturer)
- hold through material – no socket (only THERMOtec®)

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HEBA-STARR, THERMOtec®, THERMOtec®-OPTIMUM

Slimclick Micro-tube earmold

- we produce Micro-tube earmolds for all manufacturers
- available in all shapes (standard: foil plug with pull thread)
- with FlexFit socket for easy insert and change of the Micro-tube
- hold through material – no socket (only THERMOtec®)

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Slimfix Micro-tube earmold

- if it is not possible due to the volume or angling of the ear canal to accommodate the Micro-tube with socket
- the Micro-tube is glued to the earmold
- available in all shapes (standard: plug with pull thread)

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ERG0tec

- free and easy wearing feeling
- especial developed for RIC-receiver and Micro-tube systems
- proven for RIC-receiver or Slim-tube fitting with problematic ear canal course
- allows a flexible adjustment of the RIC-receiver
- only open to the wall of the ear canal
- follows the movements in the ear canal
- also suitable for influences of sound occlusion
- available in all shapes
- our patented THERMOtec® material is proven for this design

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FINOtec

- wafer-thin and deep-seated foil made in THERMOtec®
- can be used variable
- for RIC-receiver (Phonak, Widex, new Oticon Alta) and all types of Micro-tube devices
- hold through material – no socket (only THERMOtec®)
- it can be changed between the systems without processing
- available in all shapes

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Earmold silver-plated

- all shapes possible
- also part-plated possible (just surface with skin contact)

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Material: Titan (Vollguss)

Earmold Titanium

- all shapes possible
- very skin-friendly. Foil shape earmold for low specific weight. Please just send sufficient test-worn-earmolds. For production we need the earmold AND the impression. You will get back both after production.

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LASER-marking for earmolds

If you wish, we label your earmolds with a laser of the newest generation.

Optional date, name, or left/right labeling. Please contact us.

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Additional equipment BTE Earmolds

- Additional vent
- External groove
- Sealing bulge
- Horn tube
- Horn hole
- Slim-tube
- Bakke horn
- Acoustic tube
- Acoustic tube with expanding ring
- Elbow Ø 2,5 mm or Ø 3,0 mm
- Vario-venting

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Surface refinement

- Polished
- Polished / matted
- Matted
- Silver-plating
- Gold-plating
- Silicone-coating
- Glazing (anti-allergy coating, free for every rigid earmold)
- NANOlac®-coating = antibacterial finish for earmolds and hearing aids

HEBA-STARR..... hart	HEBA-STARR.....hard
HEBA-KOMBI..... hart / weich	HEBA-KOMBI.....hard / soft
HEBA-LP/H..... hart (lichthärtend)	HEBA-LP/H.....hard (light curing)
HEBA-SILICON..... weich (60/40 Shore)	HEBA-SILICONE.....soft (60/40 Shore)
HEBA-AQUA Catamaran..... weich / schwimmfähig	HEBA-AQUA Catamaran.....soft / floatable
THERMOtec®..... Körpertemperatur weich (ca. 70 Shore)	THERMOtec®.....body temperature soft (approx. 70 Shore)
..... Raumtemperatur hart (patentiert)	 room temperature hard (patented)
THERMOtec®-OPTIMUM..... Körpertemperatur weich (ca. 50 Shore)	THERMOtec®-OPTIMUM.....body temperature soft (approx. 50 Shore)

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Unbreakable material, well tolerated. A high-quality hot curing resin earmold material. A dermatological expertise confirms the excellent tolerability.

Colors:
pink-clear, clear, red, blue, yellow, green, black

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Unbreakable material, well tolerated. A high-quality hot curing resin earmold material, but with a soft auditory canal part.

Colors:
pink-clear, clear, red, blue, yellow, green, black
auditory canal part:
pink-clear or clear

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Farben:
rosa-transparent, transparent, beige, flesh, rot, blau

One-component plastic based on acrylate. Accurate fit, especially suitable for ITE shells.

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60 Shore:
transparent, rosa-transparent, rot, blau

High-quality earmold silicone, extremely tensile resistant and permanently elastic.

Colors:
40 shore:
clear, pink-clear, red, blue, yellow, green, white, black

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Floatable special silicone, permanently elastic, extremely tensile resistant and long-term color-stable.

Colors:
35 shore
red, blue, yellow, green, orange, white, pink, violet

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Very skin-friendly. Foil shape earmold for low specific weight. Please just send sufficient test-worn-earmolds. For production we need the earmold AND the impression. You will get back both after production.

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A unique plastic material developed and patented by HEBA especially for earmolds. Physiologically harmless and safety-tested. Tested with excellent results for „dermal irritation“, „sensitization“ and „cytotoxicity“ according to ISO 10993-I by internationally recognized institutes.

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Hard outside the ear (room temperature) – soft in the ear (body temperature).

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Further decisive advantages:

- easy to insert
- simple to work
- no pressure pain under jaw movement
- optimal sealing
- stays elastic
- feels natural, like a second skin
- no monomers, perfectly proven for allergies.
- ITE-devices do not slip out of the ear
- soft and flexible in the ear canal
- absolutely tear-resistant
- no UV discoloration
- patented and certified according to DIN EN ISO 13485

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Colors:

pink-clear, clear, red, blue, yellow, green, black (matted)

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Also suitable for all types of RIC-receiver and for open fitting

- natural wearing feeling with best skin-compatibility
- hard at room temperature
- easy to insert

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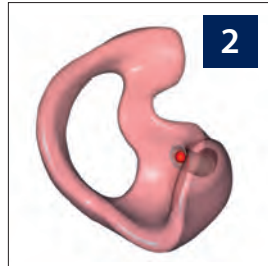
The trend-setting technologies of the CAD-production of earmolds make new scales with respect to service and fitting accuracy possible.

To let our patented material THERMOTec® profit from these advantages, we have developed a process which makes it possible for us to integrate THERMOTec® into this modern computer generated production process.

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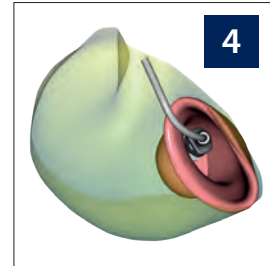
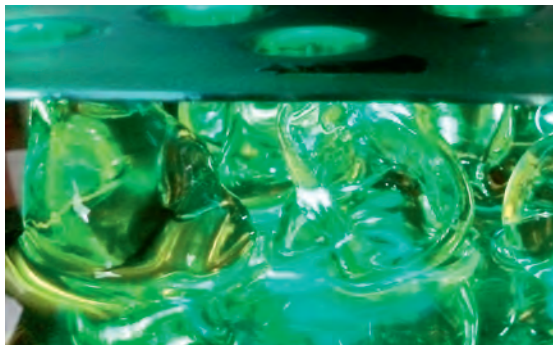
From this, the following advantages arise:

- very high fitting accuracy
(production based on scanned impression)
- archiving of data (in case of loss or defect no new impression is required)
- individual and functional requirements can be better implemented on the computer
- consistently high quality
- best results at the production of RIC-receiver and Micro-tube earmolds

Thereby, THERMOTec® and THERMOTec®-OPTIMUM will fulfill the highest demands in the future.

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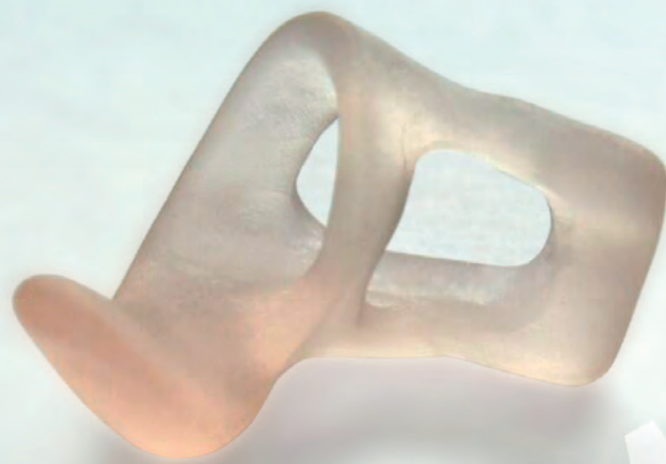
Glitter

for surface and to mix into the material, colors on request

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Rhinestones

in 3 different sizes, colors on request

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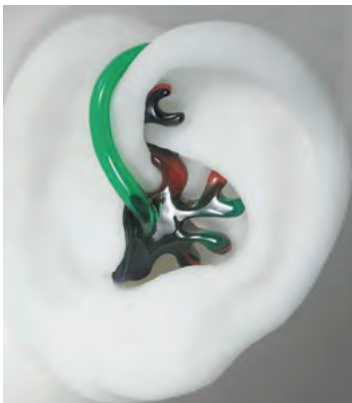
Design earmold

- with an extended claw
- 3-color marbled
- different colors are possible
- when making the impression, also mold the extended claw

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Design earmold

- with an extended claw
- 3-color marbled
- various color combinations are possible

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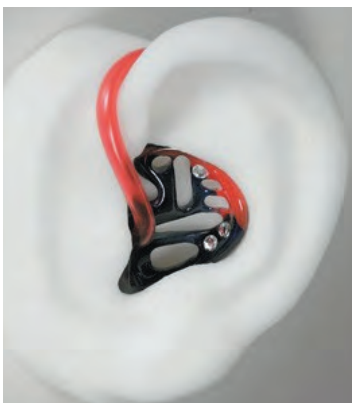
Design earmold

- only available in shell shape
- 2-color
- various color combinations are possible

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Design earmold

- with 3 crystal rhinestones
- only available in shell shape
- various color combinations are possible

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Material:
HEBA-STARR

Design earmold

- with 2 crystal rhinestones
- only available claw shape
- various color combinations are possible

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Material:
HEBA-STARR

Design earmold

- with 5 crystal rhinestones
- only available in shell shape
- various color combinations are possible

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Material:
HEBA-STARR

Design earmold

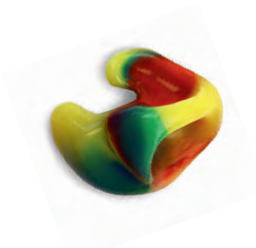
- with an extended claw
- 2-color marbled
- different color combinations are possible
- when making the impression, also mold the extended claw

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HEBA has a number of information brochures for you. Just write an e-mail to info@heba.de or call us at **+49-6022-681600**. We will be glad to send you the required information material.

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
Your hearing ability can influence the way you get along in everyday life. Individually manufactured hearing protections are used where maximum protection and comfortable fit are required.

Our hearing protection earmolds stand out by their long service life. The insulation effect can be selected by filters for the individual application.

Filter lieferbar in transparent, rot und blau.
Filter available in clear, red and blue.

Filter	ER 25	ER 15	ER 9
SNR-Wert in dB	23	16	10
H-Wert in dB	22	14	10
M-Wert in dB	21	14	8
L-Wert in dB	21	14	8

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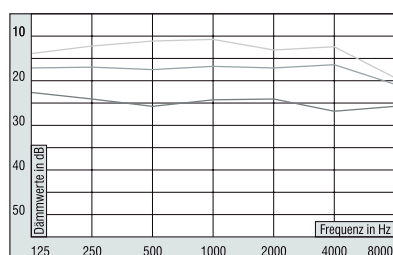ELACIN® ER 9/15/25

Individual comfort-hearing-protection

ELACIN® ER is the comfort-hearing-protection for all those who are regularly exposed to loud noises, but still have to listen carefully: musicians, sound technicians, dentists, but also concert audience and disco-guests. Optimal protection in combination with a natural sound reproduction, comfortable to wear and cosmetically attractive. Special broadband filters guarantee a constant insulation in the frequency range of 125 to 8000 Hz.

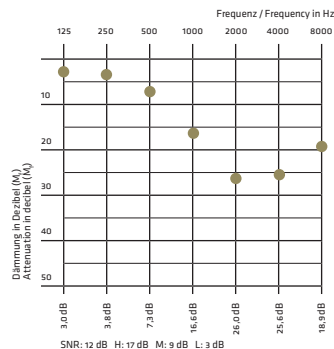
The timbre is maintained, the sound remains natural. There is no occlusion effect. ELACIN® ER is custom-made after an impression and fits permanently and comfortably in the ear — safely and precisely. In combination with the IEM-15 earphone, the ELACIN® ER earmold is perfectly suitable to be used on stage or to be connected to an MP3 player, tablet and Smartphone.

- three different filters available
- standard supply with ER 15 filters
- filter colors: red, blue (only ER15/25) and transparent
- suitable as an earmold for the IEM-15 earphone
- earmold made of soft, durable silicone
- color code for right/left
- very comfortable to wear
- easy to clean, long service life

Tested according to DIN EN-352-2, type examination

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SHOTKILLER®

Individual comfort-hearing-protection

- effective protection against impulse noise
- nonlinear insulation effect
- Applications: Leisure
- additional HEBA ribbon possible


Attention: This filter has no PSA-approval.

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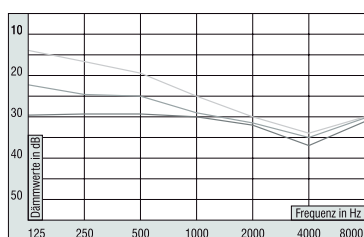ELACIN® BIOPACT

Individual comfort-hearing-protection

The patented insulation filter ELACIN® Biopact, integrated into the individual custom-made silicone earmold guarantees an optimal selective insulation in a broad frequency range. Thanks to its perfect fit the earmold does not cause any pressure in the hearing canal. The integration of the filter element prevents a reduction of the wearing comfort by a closing effect. Conventional hearing protections mainly insulate in high frequency ranges. ELACIN®-Biopact filters are broadband filters that reach a high insulation effect even in lower frequency ranges.

Filter:
ML01 black
MM02 brown
MM12 white

Fields of application: Universal private and professional use due to filter selection. Due to its small design it is also suitable for motorcyclists.

- three different filters available
- earmold made of soft, durable silicone
- color code for right/left
- very comfortable to wear
- easy to clean, long service life

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Ear-protection for bikers
With ELACIN biopact-filter MM 02.
Standard in pink-clear.
Also with ribbon deliverable.

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
ELACIN® ClearSound
Individual comfort-hearing-protection

ELACIN® ClearSound protects the hearing accurately: graded according to the noise surrounding, open for understanding speech and warning signals, for daily use. Ideal application: workplace in crafts and industry, do-it-yourself, traffic, private sector. Four different acoustic insulating filters are available with ELACIN® ClearSound — suitable for every application.

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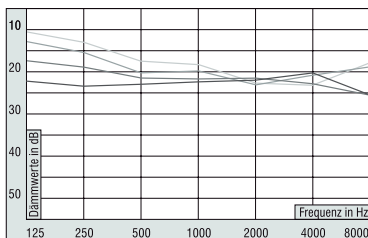The filter technology especially developed prevents excessive sound insulation. ELACIN® ClearSound protects your ears in the best possible way. ClearSound can easily be worn during communication. Depending on the selected filter good insulation results are achieved at low frequencies up to high frequencies. The insulation filter is completely integrated into the earmold and is fixed firmly.

- good protection at all frequencies
- four different filters available
- earmold made of soft, durable silicone
- color code for right/left
- very comfortable to wear
- easy to clean, long service life

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Filter:
RC 15 beige
RC 17 grau
RC 18 braun
RC 19 schwarz

Tested according to the Directive 89/686/EEC
(Personal Protective Equipment) and
DIN EN 352-2: 2002

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IN-EAR-MONITORING – professional sound and very comfortable to wear

The I-E-M earmold is the ideal product for those who love music and place great importance in quality. The custom-made monitor earmolds ensure an almost invisible and aesthetic connection between the headset and the ear.

Other typical applications:

- listening to music on an MP3 player, tablet and smartphone
- headset for safe telephoning while driving
- during sports such as Nordic walking, inline skating, skiing or jogging

I-E-M earmolds are individually adapted to your ear – their wearing properties are much better than those of conventional earphones, they remain safely in your ear and need not be replaced regularly.

Depending on the application, we use soft silicone or the absolutely skin-compatible, flexible **THERMOTEC®**.

Custom-made I-E-M earmolds are available in many different colors and varieties for all common types of earphones.

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ELACIN® ER 15 IN-EAR-MONITORING

- special receiver for an individual hearing protection
- as an option the IEM-15 receiver can be replaced by the ER filter.

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21002
21102

Ø 3 mm **PP**-Hornschlauch,
transparent
Ø 4 mm **PP**-Hornschlauch,
transparent

Ø 3 mm **PP** Horn tube clear
Ø 4 mm **PP** Horn tube clear

28070
28075

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Bakke-Horn for hard earmolds

21500

Spreizring

Expanding ring

21800

Ankerteil

Expanding ring seat

21900

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
Metal expanding coupling gold-plated

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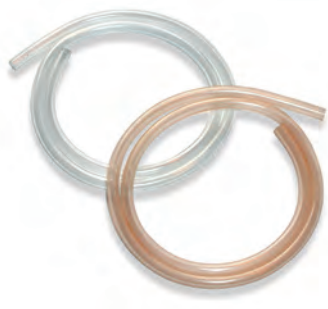**Acoustic tube DRY-tube,
curved**
Ø 2,0 x 3,0 mm clear

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Tube spiral, winded
for presenter, policeman etc.

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For a simple log-in of the RIC-receiver

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Soft canal tips standard
3 sizes:
small
medium
large

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Hucki – Retainer kit

Simple and practical facility to hold BTE hearing aids safely in place. Additionally battery case or volume control can be covered by the loops. The tubing can be individually adapted to the size of the ear.

contents:
12 Huckies in different colors
50 cm tube
10 pins

Hucki + Clip blau

Hucki + Clip rot

Hucki + clip blue

Hucki + clip red

27009

27008

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Inh. 10 Stück
hautfarben
rot
blau
braun
rosa-transparent

Hucki – Refill kit

contents: 10 pieces
flesh
red
blue
brown
pink-clear

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PUROCLEAN®
Cleaning tablets
carton with 20 tablets

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Air blower
with chrome-plated metal canula

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HEBA®-THERM
electrical drying cushion

Product advantages:

Dries hearing aids and earmolds in an absolutely reliable and homogenous way. The only water-proof drying cushion worldwide equipped with a protective conductor. Can be used in a wet or damp environment without danger. Absolutely tear-proof plastic cover and cable connection. Overheating protection. Equipped with a zipper for easy insertion and safe storage. Drying time: 1 till 2 hours

Technical description:

Surface: PVC, flame protected
Design: protection class I, VDE-GS and SEV tested.
Power consumption: 4,5 Watts, 110 or 230 V
HEBA®-THERM increases the operability and service life of your hearing aid.

HEBA®-THERM is available in three color-designs:
blue
green
tabby

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Timer for HEBA®-THERM drying cushion

electromechanical version
simple and reliable

runtime: 1 minute to 1 hour
with TÜV/GS/CE approval mark

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HEBA®-THERM opti-TIME electrical drying cushion

The drying cushion with the distinct advantages

- switches off automatically after 3 hours
- 12-Volt — international usage
- overheat protection
- zip for easy use, easy washable cover

Specifications:

- surface: PVC non-flammable
- AC adaptor plug 100-240 V AC
- power input: 4.5 Watt / 12 Volt DC
- CE mark
- drying temperature: approx. 40 °C (delicate drying)
- runtime of Timer 3 hours (automatic shutdown)

You can receive the HEBA®-THERM opti-TIME also in the practical design carrier bag in different arrangements — ideal for the drying of your hearing aid if you are on the move. The pockets can be equipped individually. Please talk with us.

HEBA®-THERM opti-TIME is available in three color-designs:

blue
green
tabby

HEBA®-THERM opti-TIME
with private label ex 100 pieces

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(LxBxH): 132 mm x 90 mm x 37 mm

DRY-STAR-UV 2 Drying station

Moisture and ear wax can impair the function of hearing systems and cause failures.

The DRY-STAR-UV 2 offers optimal care. Comfort dry at 55°C extends life time significantly.

Ultraviolet light (253,7nm UV-C) eliminates 99,9 % of bacteria, fungi or viruses.

This clinical cleanliness prevents ear infections risks and protects both your hearing systems as well as your health.

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Drying and Cleaning set / BTE
I HEBA®THERM drying cushion
I HEBACLEAN® cleaning powder
(approx. 50 cleaning bath with active oxygen)
I air blower
I cleaning tub
in design-carrier-bag

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Drying and Cleaning set / ITE-SB
I HEBA®-THERM drying cushion
I bottle cleaning spray with atomizer and brush
10 cell tissues
in design-carrier-bag

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Drying and Cleaning set / ITE-RT
I HEBA®-THERM drying cushion
25 cleaning tissues in pocket dispenser
10 cell tissues
in design-carrier-bag

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**PUROCLEAN®
Cleaning spray**
30 ml
with atomizer and brush

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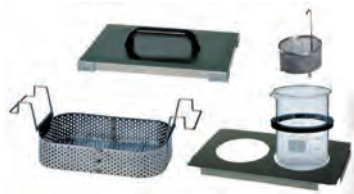
Clean box

A good way to start professional ultrasonic cleaning.

- suitable for the cleaning of earmolds, spectacles and instruments
- in addition, it is possible to use a different cleaning media in each of the beakers
- easy and comfortable to operate, pre-programmed cleaning time
- good visible LED-display
- practically: the cover can also be used as basket holder (drip drying)

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netto: 450 g

U-Sonic Ultrasonic cleaner

Electrical Cleansing station for earmolds

Your earmolds, hearing protection and slim tubes will be perfectly cared by comfort cleansing, the life span increased, the risk of failure and repair costs significantly reduced. Easy handling by touch on a sensor button, everything is handled by processor fully automatic.

The result — hygienic cleanliness, Germs are reduced and health risks is minimized.

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Addition-vulcanizing impression materials

Precise impression taking by completely non-shrink addition-vulcanizing silicone material. Components A and B mixed 1:1. Easy working.

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HEBA-FORM A+B

kneadable silicone impression material
non-sticky and smooth
final hardness: approx. 35 Shore A
unit: 2 x 300 g

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HEBA-FORM A+B

kneadable silicone impression material
non-sticky and smooth
final hardness: approx. 35 Shore A
unit: 2 x 500 g

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HEBA-FORM A+B

kneadable silicone impression material
non-sticky and smooth
final hardness: approx. 35 Shore A
unit: 2 x 1.500 g

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HEBA-FORM A+B Ultra-Light

kneadable impression material
silicone, non-sticky and very smooth.
„Ultra-Light“ is particularly suitable for impression
taking for CIC/IdO devices due to its soft consi-
stency
final hardness: approx. 25 Shore A
unit: 2 x 500 g

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HEBA-FORM A+B Soft

Silicone impression material in double cartridges
— high stability
— dimensionally stable
— does not flow out of the ear
— high tensile strength
final hardness: approx. 35 Shore A
unit: 8 x 50 ml double cartridges

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8 x 48 ml Doppelkartuschen

HEBA-FORM A+B CIC

Silicone impression material in double cartridges.

- high stability
- dimensionally stable
- does not flow out of the ear
- high tensile strength

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HEBA-FORM A+B KIDS

Silicone impression material for kids and teenagers (also for small/narrow auditory canals)

- high stability
- low viscous, smooth consistency
- extremely tear resistant, highest elastic recovery from deformation
- skin-friendly & biocompatible
- child-friendly: extra short time in the ear (2 min.)

Final hardness: approx. 25 Shore A

Unit:

4 x 10 ml Mini-cartridges

1 x reusable plunger

8 x mixing canulas for Mini-cartridges

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orange

HEBA-AQUA Catamaran

Floatable Silicone for direct fitting of earmolds for noise and water protection, tear proof. Also applicable with the indirect method in the lab.

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Impression syringe
double piston system,
with transparent cylinder
Ø 3 mm
Ø 4 mm
Ø 5 mm

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Spare piston
for impression syringe

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Mixing spatula
from stainless steel
with wooden handle

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rechts
links

Demo ear

Helpful for practice in taking impressions —
doesn't stick together by use of silicone
impression materials

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Impression plugs foam

unit: 100 pieces with thread
conical, pink
cylindrical, white

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Impression plugs cotton

unit: 50 pieces with thread
small
medium
large

36505
36507
36509

Pinzette, gebogen

Tweezer, curved

60100

Pinzette, Bajonett

Tweezer, bayonet

60200

Schere, Edelstahl

Spitze stumpf-stumpf
Spitze spitz-stumpf

Scissors, stainless steel

tip rounded-rounded
tip pointed-rounded

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complete with 4 reusable funnels and
5 disposable funnels each of the diameter
2,5 mm and 4 mm, battery operated,
available in:
black
blue

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HEINE Reusable funnels

1 set, all sizes
2,4 mm, 1 piece
3,0 mm, 1 piece
4,0 mm, 1 piece
5,0 mm, 1 piece

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HEINE Disposable funnels
Ø 2,5 mm unit: 50 pieces
Ø 4,0 mm unit: 50 pieces

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Spare parts for MINI 2000 and Ear light are still available!

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schwarz
blau

LUXAMED® Otoscope LED
Best handicraft production is combined with the most modern technology. One convincing example: with unique illumination, an innovative LED-Ring with a nevertheless free field of vision and optimum illumination in the application area. No bulb changes necessary.

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schwarz
blau

LUXAMED® Otoscope LED / USB

The LUXAMED Otoscope LED / USB with integrated charging technology makes the difference. Easy charging at a power supply or a computer without being bound to a charging station.

incl. 20 disposable funnels
(10 x 2,5 mm + 10 x 4,0 mm)
and recharger with USB-plug (adapter EU/UK/
US)

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schwarz

LUXAMED® Ear light

Quality workmanship, lamp is a white LED, battery operated

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LUXAMED® Spare acrylic top for ear light

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Scalpel (one-way)
contents: 10 pieces single,
sterile packed

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WWW.HEBA.DE

NEGATIVFORM HERSTELLUNG MANUFACTURING OF NEGATIVE FORMS

NEGATIVFORMHERSTELLUNG MIT DUBLIERMASSE	S. 58
MANUFACTURING OF NEGATIVE FORMS WITH DUPLICATING MATERIAL	p. 58
NEGATIVFORMHERSTELLUNG MIT DUBLIERSILICON UND GIESSMASSE	S. 59
MANUFACTURING OF NEGATIVE FORMS WITH DUPLICATING SILICONE AND CASTING MATERIAL	p. 59
NEGATIVFORMHERSTELLUNG MIT GIPS	S. 62
MANUFACTURING OF NEGATIVE FORMS WITH PLASTER	p. 62

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Dubbing material

especially for UV-light technique for producing
negative forms, reusable

processing temperature:
45 – 47 °C (113-116,6 °F)
melting temperature: 93 °C (194 °F)
6 kg

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Casting forms, transparent

with removable bottom
with removable bottom and cover
10 pieces

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Cuff forms

Ø 26,0 mm - 50 pieces
Ø 35,0 mm - 25 pieces
Ø 40,0 mm - 25 pieces
Ø 50,0 mm - 25 pieces

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Base parts for cuff forms

Ø 26,0 mm - 1 piece
Ø 35,0 mm - 1 piece
Ø 40,0 mm - 1 piece
Ø 50,0 mm - 1 piece

46604

46608

46610

46612

Automatik Heizplatte

1500 Watt, stufenlos, feinfühlige Tempera-
turregelung

Automatic heating plate

power consumption: 1500 Watt
temperature continuously adjustable

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Duplicating silicone transparent

transparent addition vulcanising silicone
for producing negative forms.
mixing ratio: 1 : 1
final hardness: approx. 15 shore

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Farbe:
Komponente A = Gelb, 1 kg
Komponente B = Blau, 1 kg

Casting material

addition vulcanising for producing
negative forms.
mixing ratio: 1 : 1
final hardness: approx. 18 – 20 shore

color:
component A = yellow, 1 kg
component B = blue, 1 kg

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Silicone dosing pump

to remove the silicone quickly and clean
from the 6-kg-container

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Pressure pot for polymerization

for cold-curing resin

with air pressure hose including fill and inlet valve

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Innen Ø 100 mm

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Polypot pressure pot

Pressure polymerization pot for cold-curing resin in water bath. Pressure will be achieved by turning the spindle up to 2 bar, with manometer.

works without compressed air connection

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Surface cleaner
for cleaning the surface of grease, dust, silicone
film etc., before using of modeling paste or
lacquer

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LP-cleaner
for second polymerization of the sticky
surface (inhibition layer) which emerges during
the light polymerization due to the effect
of oxygen.

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HEBA-LG light-curing unit Type I
— with integrated timer (minutes and seconds)
— for up to 12 earmolds in one operation
— universally applicable for ITE-shells, earmolds,
lacquer and repair work

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Spare parts:
set of reflecting foils compl. for Type I
plastic plate clear for Type I

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HEBA-LG light-curing unit type 1 plus N 2

Modified light curing unit only for repolymerization works. Same model as Type 1 (Art.-Nr. 44000), but equipped with gas supply (incl. 5 m gas hose). Repolymerization is carried out in a work box filled with nitrogen.

Formation of an inhibitory layer is prevented since no oxygen influences the polymerization process.

This eliminates the usual 3rd polymerization in LP-Cleaner.

Nitrogen is harmless if used properly and available in specialist shops. The following accessories make it necessary for the modified LG plus N 2 device and should not be missing in the complete equipment.

size:
W 280 x D 300 x H 210 mm
weight: ca. 9 kg
power consumption: 250 W
110 Volt type available

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Spare parts for type 1 plus N 2

pressure reducer for gas bottle incl. tube

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HEBA-LG light-curing unit Type 2

with integrated timer (minutes and seconds) for up to 6 earmolds

size:
W 280 x D 240 x H 120 mm
weight: 4,8 kg
power consumption: 80 W
also as 110 Volt type available

Spare parts:

set of reflecting foils compl. for Type 2
plastic plate clear for Type 2

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HEBA-LG light-curing unit Type 3

For repairing, curing of earmolds and laquers.
Especially suitable for use in the acousticians specialist shop.

size:
W 100 x D 240 x H 100 mm
weight: ca. 1,4 kg
power consumption: 30 W
110 Volt model available.

Spare parts:

set of reflecting foils, compl. for Type 3

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Turning motor for light-curing unit Type 3

small turning motor for coating
battery operated
size: 80 x 40 x 85 mm

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Turning motor – electrically – for light-curing unit Type 3

powerful turning motor, used for coating in continuous operation
electrical supply 220 V
size: 80 x 40 x 85 mm

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HEBA-LG light-curing unit Type 4

— high performance unit for coating and repairing up to 25 earmolds in one process
— automatic switch off after 60 minutes
— built-in ventilator

size:
W 220 x D 300 x H 110 mm
weight: 5,2 kg
power consumption: 120 W

Spare parts:

set of reflecting foils compl. for Type 4
varnishing drawer for Type 4

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Light tubes for LG UV units
blue light

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Light-curing modeling set
for non-allergic coating, repairing, finishing
earmolds etc.

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consisting of:
light-curing unit Type 3 (230 V), LP/H
modeling paste (clear, beige, red, blue), LP/H
lacquer 50 ml, LP/W lacquer 50 ml, surface
cleaner 250 ml with hand duster, LP-cleaner
100 ml, tub clear, modeling instrument, Nylon
tube as lacquering aid, digital timer, Cyana-
crylate glue with 10 dosing tips, 10 curved
acoustic tubes 2.0 x 3.1 mm

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with battery operated turning motor
with electrical turning motor

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Cold-curing resin

The polymerization is carried out under pressure in the pressure pot, polypot pressure pot or Micropol 100.

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HEBA-ACRYL cold-curing resin

A high-quality cold-curing resin on the base of methyl methacrylate, no amine or cadmium. The earmold material can be cast easily. It has a perfect color stability after polymerization and is excellently tissue-compatible.

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Hot-curing resin

The polymerization is carried out under pressure and heat in the microwave oven.

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HEBA-ACRYL hot-curing resin

A high-quality self and hot-curing resin on the basis of methyl methacrylate, no amine or cadmium. The earmold material can be cast easily. It has a perfect color stability after polymerization and is excellently tissue-compatible. This hot-curing resin was developed especially for the short-term polymerisation.

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Farben:
rosa-transparent
transparent

HEBA-FLEX hot-curing resin

Acryl hot-curing resin for soft earmolds or combined earmolds from HEBA-acrylic hot polymerizate (Art.-Nr. 50800) and HEBA-FLEX. Acryl hot-curing resin liquid for processing with HEBA-FLEX powder ensures a soft setting of the material.

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Inhalt: 25 ml

blau
rot
gelb
grün
schwarz

Color concentrate for cold- and hot-curing resin

depending on the added amount intensive or transparent colors possible.

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silber
gold
blau

Glitter for earmolds

for mixture with cold-curing resins and UV-light material

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Mixing spatula

for mixing powder and liquid
stainless steel

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70 ml gross
30 ml mittel
5 ml klein

Mixing cup

of flexible silicone material with a practi-
cally pouring aid at 30 ml and 70 ml

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Pressure pot for polymerization

for polymerization of cold-curing resin with
air pressure hose including fill and inlet valve
capacity: 4 l
inside: Ø 200 mm
height inside: 180 mm

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Polypot pressure pot

Pressure polymerization pot for cold-curing
resin in water bath for up to 4 earmolds.
Pressure will be achieved by turning the spind-
le up to 2 bar, with manometer.
Works without compressed air connection.
inside: Ø 100 mm
height inside: 65 mm

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Polymax-I

Polymax units are machines for pressure-polymerization of powder/liquid resins which polymerize in a temperature area from 30 °C to 95 °C under pressure within 90 minutes.

Features:

1-90 minutes timer / 5 bar; variably adjustable; automatic safety-sealing-system; control by digital indication of time, temperature and pressure through manometer; pressure pot made of special coated aluminium; external housing made of hard-wearing PU; housing cover made of stainless steel; maintenance-free technique; water drain valve; when heating is switched off, unit is suitable for compaction pressure.

Technical data:

H x W x D: 230 x 290 x 330 mm
pressure pot: Ø 160 mm
pressure pot depth: 110 mm
weight: 10.2 kg
power consumption: 450 W at 230 V
pressure min/max: 3/5 bar
temperature range: up to 95 °C

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Microwave technique:

Micropol 100 complete

Pressure polymerization flask for the production of acrylic and silicone earmolds using microwave energy.
for 10–12 impressions, can be cased in one flask.
high-grade steel, hard-chrome plated.

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Accessories for Micropol 100

embedding set for Micropol 100
membrane foils, 100 pieces
box for membrane foils
closing ratchet
screwdriver for valve to release air pressure
O-ring for Micropol 100
valve complete
screw, spare part for Micropol 100
safety glass
rubber gasket grey

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Silicone – cold vulcanization

Addition vulcanizing silicone for the fabrication of soft BTE and hearing protection earmolds. Very tear resistant, permanently elastic. Highest wearing comfort due to soft-flexible fitting, skin-friendly. A-silicone, light bodied, extended processing time (approx. 3:30 min.) for mass-production in the lab.
Final hardness: approx. 40 / 60 Shore A

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HEBA-AQUA Catamaran

Floatable silicone for adapting of earmolds for noise and water protection. Tear resistant. Also to work in the laboratory.
Final hardness; approx. 35 Shore A

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unit: 8 x 50 ml double cartridges

blau/rot
colormix
gelb
grün
orange

blue/red
colormix
yellow
green
orange

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100 ml
500 ml
1000 ml

NANOLac®

antibacterial surfaces for hard earmolds and hearing aids.

NANOLac® makes use of the natural bactericidal and fungicidal effect of „active silver“ (AgPure). The external layer of every nanoparticle consisting of silver ions develops its effect even in very low quantities. NANOLac® does not have any toxic effects, since the release of active silver takes place in a controlled way.

- NANOLac® has a prophylactic effect against inflammations in the ear
- NANOLac® is antibacterial and fungicidal
- NANOLac® encourages the natural microclimate balance in the ear
- NANOLac® prevents itching

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100 ml
250 ml
500 ml
1000 ml

LP/H – lacquer

light-curing lacquer for hard earmolds

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100 ml
250 ml

LP/W – lacquer

light-curing lacquer for soft earmolds

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10 ml
250 ml
500 ml

HEBA-CON – lacquer

air drying, for silicone earmolds (with Toluol)

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100 ml
250 ml
500 ml

Lacquer H/II (silicone)

hot vulcanizing silicone lacquer for finishing the surface of silicone earmolds (free of Toluol).

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Inhalt: 20 g Tube

HEBA-CON adhesive (silicone)

Adhesive for tubes and elbows in silicone earmolds.

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Inhalt: 10 g

Super glue

one component glue based on cyanacrylate

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Inhalt: 10 g


Cyanacrylate glue

in connection with primer for durable bonding of tubes and elbows into silicone earmolds.

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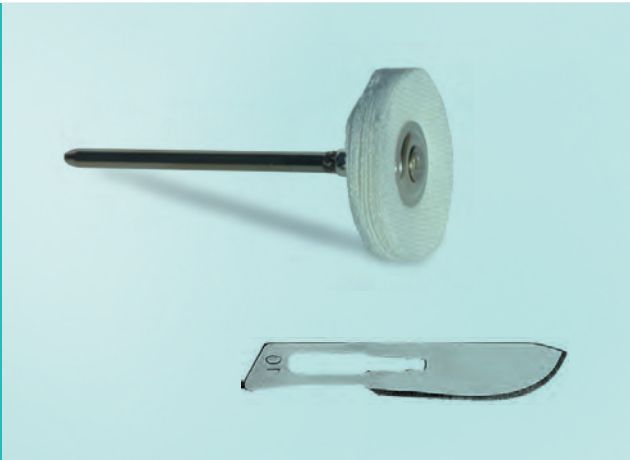Tubing expander
stainless steel, chrome plated, curved

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70 ml gross
30 ml mittel
5 ml klein

Mixing cup
of flexible silicone material with a practically
pouring aid at 30 ml and 70 ml

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Scalpel (one-way)
contents: 10 pieces single, sterile packed

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
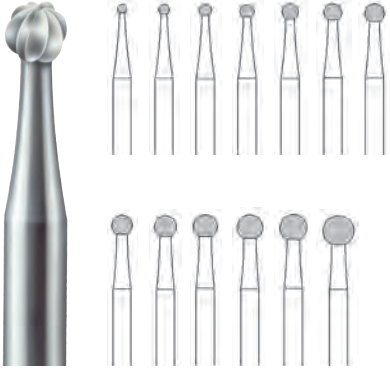
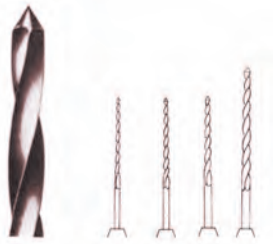
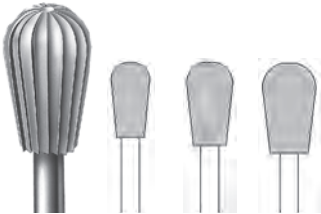
Cutters and drills

all instruments have a shank diameter of 2,35 mm

Bohrer		drills	
	Rosenbohrer	Ball burs	
	Ø 1,2 mm Ø 1,4 mm Ø 1,6 mm Ø 1,8 mm Ø 2,1 mm Ø 2,3 mm Ø 2,5 mm Ø 2,7 mm Ø 2,9 mm Ø 3,1 mm Ø 3,3 mm Ø 3,5 mm Ø 4,0 mm	Ø 1,2 mm Ø 1,4 mm Ø 1,6 mm Ø 1,8 mm Ø 2,1 mm Ø 2,3 mm Ø 2,5 mm Ø 2,7 mm Ø 2,9 mm Ø 3,1 mm Ø 3,3 mm Ø 3,5 mm Ø 4,0 mm	66000 66100 66200 66300 66400 66401 66500 66550 66600 66601 66650 66700 66800
	Spiralbohrer	Twist drills	
	aus HSS Material mit Größen-Kennzeichnung Ø 0,6 mm Ø 0,8 mm Ø 1,0 mm Ø 1,2 mm Ø 1,4 mm Ø 1,6 mm Ø 2,0 mm Ø 2,35 mm Ø 2,5 mm Ø 2,9 mm	Carbide material with size marking Ø 0,6 mm Ø 0,8 mm Ø 1,0 mm Ø 1,2 mm Ø 1,4 mm Ø 1,6 mm Ø 2,0 mm Ø 2,35 mm Ø 2,5 mm Ø 2,9 mm	67500 67600 67700 67900 67800 67901 68000 68005 68050 68001
Fräsen		cutters	
	Fräsen, birnenförmig	Cutters, pear	
	Ø 5,0 mm Ø 6,0 mm Ø 7,0 mm	Ø 5,0 mm Ø 6,0 mm Ø 7,0 mm	62500 62600 62700
	Fräsen, knospenförmig	Cutters, pointed	
	Ø 5,0 mm Ø 6,0 mm Ø 7,0 mm	Ø 5,0 mm Ø 6,0 mm Ø 7,0 mm	63500 63600 63700

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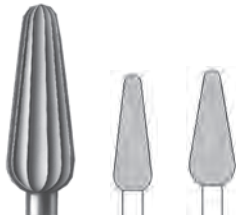Mandrel for sandpaper
cylindrical
conical

Artikel-Nr.
Order No.

70900
71000

Sandpapier
50 m-Rolle
Körnung 320
Körnung 120

Sandpaper
50 meter on reel
grain size 320
grain size 120

71400
71500

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Mandrel
for polishing discs or soft discs

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klein: Ø 5 mm, 150 Körnung, 10 Stück

Grinding caps and holders

special grinding tool for the final finishing, suitable for all kind of silicone materials.

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grinding caps large, round
grinding caps holder large, round

grinding caps small, round
grinding caps holder small, round

grinding caps large, pointed
grinding caps holder large, pointed

grinding caps small, pointed
grinding caps holder small, pointed

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
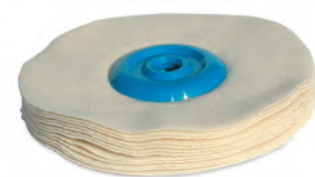
Polishing buffing wheel / Highgloss finish

made of flannel material
for polishing motor Ø 100 mm

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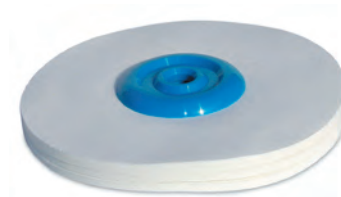
Polishing buffing wheel

made of cotton material
for polishing motor Ø 100 mm

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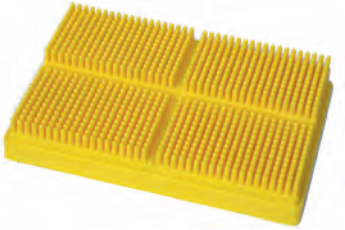**K-POWERgrip unit
permanently mounted**

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Laborgerät Perfecta 300/600

The laboratory advice Perfecta 300/600 is
suitable for the precise machining of all with
the motor handpiece processed materials by
laboratory and audiologist.

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Perfecta 300: 1.000 – 40.000 r./min.

under table unit with motor handpiece
tabletop unit with motor handpiece
knee operated unit with motor handpiece

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**Dust extraction system,
SMARTair, Evo, permanently
mounted**

Quiet, change of filter quick and easy. Suction
intensity of 20 to 32 l/sec. Optimum protection
against micro dust
3-filter-system: combined use of filter-bags,
micro-filter and carbon-filter possible. Fits in
any installation situation

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Dust extraction system SMARTair, Evo, mobile

Quiet, change of filter quick and easy. Suction intensity of 20 to 32 l/sec. Optimum protection against micro dust. 3-filter-system: combined use of filter-bags, micro-filter and carbon-filter possible.

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Vibrator, small

This small, powerfull vibrator is indicated for the fast and bubble-free pouring of forms. The speed of 6000 vibrations per minute can be switched to low or high intensity. The unit is maintenance free and can be used for continous operation without loss of power. The vibrations are uniform on the whole platform table. The rubber cover can be easily removed and cleaned.

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Vibrator, large

This vibrator distinguishes itself for its variable intensity control for both vibration speeds. The very strong swinging elements allow the fast and bubble-free pouring of forms at 3000 and 6000 vibrations per minute. The resistant rubber cover of the flatfom table is easy to clean. The suction cup feet and the high own weight make the maintenance-free unit low-noise and stable on the bench.

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Technical data:

W / H / D: 240 / 110 / 150 mm
weight: 5,5 kg
rated voltage: 230V
cycles: 50/60 Hz
power consumption: 40W
speed/min: 3000/6000 Hz

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Polishing motor

powerful and sturdy polishing motor for wet and dry polishing, maintenance free.

with one spindle
one grinding wheel holder
height of spindle: 150 mm
power consumption: 200 W
speed: 1500 / 3000 r./min.
weight: 14 kg

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Polishing trough

Made of flexible hard rubber. With height-adjustable protection screen made of safety glass.
For collecting polishing abrasive and dust. Connection to dust extraction system possible.

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Schleifscheibe für Singlespeed

Trimmer, singlespeed

The powerful 750 Watt motor can be operated at 3000 r./min. The housing is corrosion resistant, can be easily opened and the water supply can be individually set. The coating of the high-performance abrasive disc can be replaced.

Technical data:

width: 305 mm
height: 335 mm
depth: 430 mm
weight: 19 kg
rated voltage: 230 V
cycles: 50/60 Hz
power consumption: 750 W
RPM: 3000 r./min.

Accessories:

disc with grinding coating for singlespeed

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Trimmer, doublespeed

The powerful 1000 Watt motor can be operated at 1500 or 3000 r./min. The housing is corrosion resistant, can be easily opened and the water supply can be individually set. The coating of the high-performance abrasive disc can be replaced. The extension of the motor axis at the back side carries a polishing quick-lock spindle.

Technical data:

W / H / D: 305 / 335 / 430 mm
weight: 20 kg
rated voltage: 230 V
cycles: 50/60 Hz
power consumption: 1000 W
RPM: 1500/3000 r./min.

Accessories:

disc with grinding coating for doublespeed
spindle left

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Spannung: 230 V

Ventilette with cooling fan

The version with newly designed 2 point electronic heat control with 300 W heating power. Used for highly demanding conditions requiring exact temperature repetition.

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standard hearing protection

SNR 36 dB

EAR-Classic soft earplugs will change the way people feel about foam earplugs, because they feel entirely different. Made of patented new foam, these plugs have a smaller cell structure and feel smoother than other foam. But that is just the beginning; the foam earplug actually reacts to the wearer's body temperature.

pocket dispenser = 5 pair
pocket dispenser = 50 x 5 pair
I pair, pocket pack
200 x I pair dispenser

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standard hearing protection

SNR 21dB

Allowing to preserve the fidelity of the original sound due to a regular noise reduction. The user can protect his ears and reduce the sound volume in a comfortable way.

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standard hearing protection

SNR 25dB

The 3-blade design fits easily in the ear canal and offers comfortable seating and consistent protection.

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1 Paar

AQUAFIT junior

standard pre-formed, lamella-shaped hearing
and swimming protection, especially for children

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1 Paar

AQUAFIT adult

standard pre-formed, lamella-shaped hearing
and swimming protection

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EarPlanes®

Standard ear plugs for pressure compensation.

A special filter ensures a regular air supply
and a continous pressure compensation.
Especially suitable for flying.

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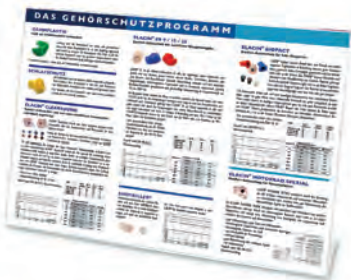
IEM-15 Earphone

Special earphone fitting to the individual Elacin ER hearing protection. The filter elements of the ear plugs can be easily exchanged with the IEM-15 earphone and your individual hearing protection becomes an in-ear-monitoring.
1 pair with 3.5 mm jack plug.

frequency range: approx. 100 – 12.000 Hz
nominal capacity: 185 mV/1 kHz (RMS)
output sound pressure: 116 dB SPL/mW bei 1 kHz
impedance: 30 Ohm bei 1 kHz

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Individual hearing-protection see chapter earmolds, page 26.

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Display „Hearing Protection“

with earmolds and demo-filter
without earmolds and demo-filter

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HEBA-service: you can count on!

Delivery service

You order before 12 a.m. we will ship your items the same day!

95 % of all our items are available immediately.

Please note that our minimum order is € 25,00.

The ultra fast delivery service!

Is it especially urgent?

Tell us your required date of delivery.

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Payment service

- payment within 30 days.
- payment within 8 days. You benefit from 2 % cash discount.

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Offer service

A call is enough!

Tell us about your individual needs. Your service consultant will directly prepare a custom-made offer for you

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Information about the transportation of dangerous goods

Under the legal regulations GGC, ADR, IMDG and IATA-DGR some of our products have to be shipped as dangerous goods. These products cannot be forwarded by regular mail.

Private parcel services are sometimes allowed to forward small quantities within Germany and Europe.

Art.-Nr. Art.-No.	Gefahrgut Dangerous goods	UN-Nr. UN-No.
45500-45625	LP/H-Lack LP/H-lacquer	1247
45700-45800	LP/W-Lack LP/W-lacquer	1866
45630+45633	NANOlac® NANOlac®	1247
46400	Oberflächenreiniger surface cleaner	1219
52405-52406	Lack H/II lacquer H/II	1206
52400+52401+ 52402	HEBA-CON-Lack HEBA-SILICONE-lacquer	1866
50200+50400	HEBA-Acryl-Flüssigkeit, Kaltpolymerisat HEBA-Acryl-liquid, cold-curing resin	1247
50900+51100	HEBA-Acryl-Flüssigkeit, Heißpolymerisat HEBA-Acryl-liquid, hot-curing resin	1247
51300	HEBA-FLEX-Flüssigkeit, Heißpolymerisat HEBA-FLEX-liquid, hot-curing resin	1993

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**08 00-4 32 26 86 oder
0800HEBAOTOPPLASTIK**

Just copy, fill in the form and send back to fax no.

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We order according to your general terms and conditions:

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HEBA-OTOPLASTIK Labortechnik u. Einrichtungs GmbH & Co. KG