

## Color concentrate

<b>units:</b>	color concentrate blue	REF: 51501
	color concentrate red	REF: 51502
	color concentrate yellow	REF: 51503
	color concentrate green	REF: 51504
	color concentrate black	REF: 51505

### Instruction for use

#### **Applications:**

Color concentrate for cold and hot-curing resin

#### **Technical Data:**

##### **Dosage:**

1 - 2 drops, depending on the color intensity

##### **Product color:**

red, blue, yellow, green and black

##### **Ordering Information:**

**Mixing spatula**  
REF 60700

**Resimix cup**  
REF 61500 large  
REF 61600 medium  
REF 61700 small

**Disposable gloves**  
REF 80415 small  
REF 80416 medium  
REF 80417 large

**Safety goggles**  
REF 80410

**HEBA-ACRYL Cold-curing resin**  
REF 50100 ff

**HEBA-ACRYL Hot-curing resin**  
REF 50800 ff

### **General product information**

Product for coloring HEBA-STARR (HEBA-Acryl hot and Cold polymerizate) and THERMOtec® raw material.

### **Special storage information**

Store in a cool and dry place.

### **Processing instructions**

Shake well before use!

Use sparingly as it is very colorful.

Earmold plastic in the mixing cup (REF 61500, 61600, 61700) according to the instructions for use mix and process.

Depending on the color intensity, put about 1-2 drops of color concentrate on the tip of the mixing spatula (REF: 60700) and stir until the color is even.

### **Attention! Protective gloves, goggles and wear suitable protective clothing.**



### **Other Information:**

Materials have been tried and tested a million times over and there are undesirable effects. Proper use is not expected. Immune reactions e.g. allergies, However, irritations cannot generally be ruled out. If in doubt, we recommend performing an allergy test before use.

